


Instruction manual:

Step 1: Turn Thermometer over and with your thumb slide forward the battery compartment door


Step 2: Once door is open place one AAA battery with the positive side up in the slot closest to the hinge of the door. Place the second AAA battery with the negative side up in the next slot. Close battery compartment door



Step 3: Ensure switch is selected for Body Temp

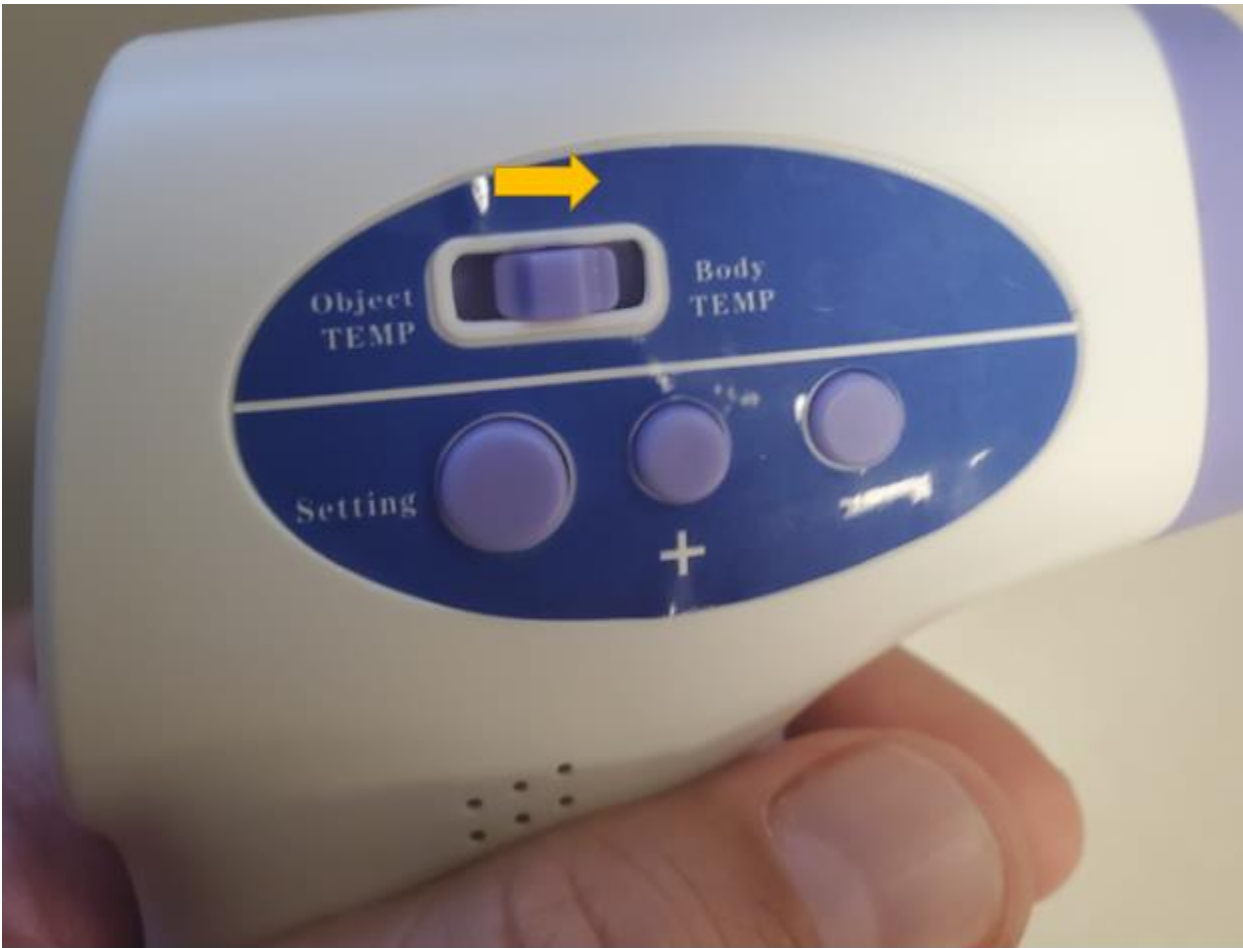

Step 4: Depress trigger for display to light up/ to change from Celsius to Fahrenheit press the setting button on the side of the unit once

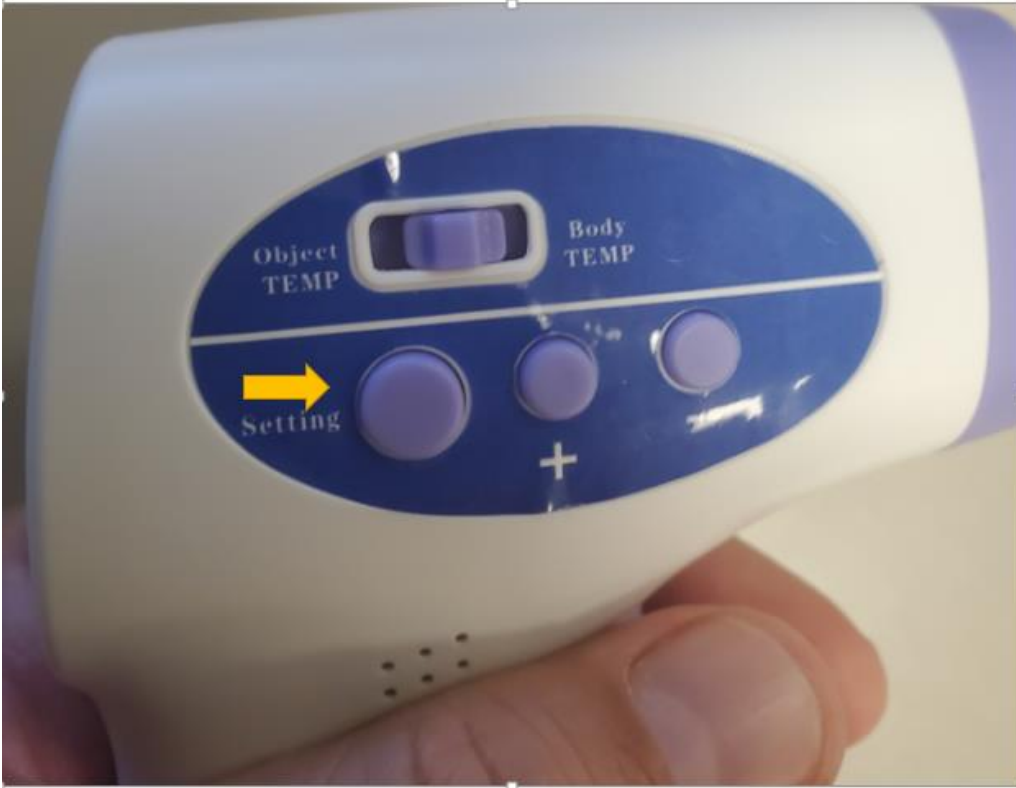

Step 6: Aim unit between 1 to 1 ½ inches from forehead/depress trigger until temp appears on screen



Step 7: Before moving onto the next person aim away from individual depress trigger and release/the screen will display “Lo” now you can take the next persons temperature

